


## Adaptor Bracket Attachment

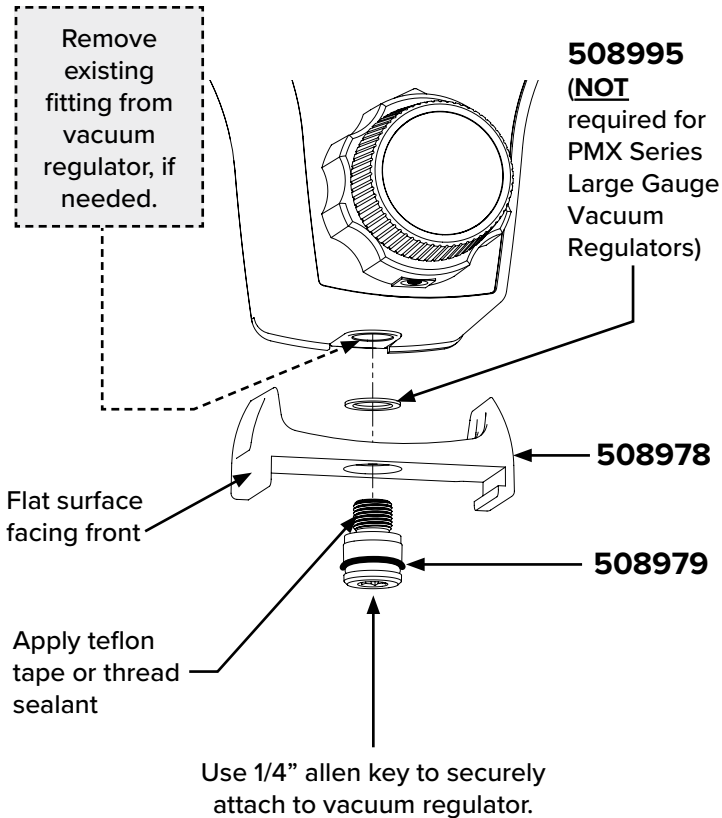

## Intended Use

The PMX900 Filtered Vacuum Regulator Trap is intended to prevent fluids/debris from entering the vacuum regulator or central suction system. The float ball is intended to indicate positive vacuum flow away from the patient toward the vacuum regulator.

Read All Instructions Before Using

This manual instructs a Professional to install and operate the Vac Trap. This is provided for your safety and to prevent damage to the Vac Trap. If you do not understand this manual, DO NOT USE the Vac Trap and contact your Provider.

### ⚠ WARNING

Use this Vac Trap only for its "Intended Use" as described in this manual.

- Only personnel instructed and trained in its use should operate this Vac Trap.
- Be sure all connections are tight and leak free.
- The Vac Trap must be in the vertical position to function.

## Troubleshooting

Problem	Probable Cause	Remedy
No vacuum from tubing nipple of trap	<ol style="list-style-type: none"> <li>1. Loose or broken connection</li> <li>2. Filter blocked</li> <li>3. Trap full of fluid</li> </ol>	<ol style="list-style-type: none"> <li>1. Check trap and connections</li> <li>2. Replace trap</li> </ol>
Fluid in bottle	<ol style="list-style-type: none"> <li>1. Fluid overflow of vacuum canister</li> <li>2. Patient tubing directly connected to trap</li> </ol>	<ol style="list-style-type: none"> <li>1. Replace trap</li> </ol>



## PMX900

Filtered Vacuum Regulator Trap

300 Held Drive  
Northampton PA 18067, USA

www.precisionmedical.com

T: (+001) 610-262-6090 • F: (+001) 610-262-6080

ISO 13485 Certified

508984 Rev 1

**Rx ONLY**


Caution! U.S. Federal law restricts this device to sale by or on the order of a physician.




## Installation and Removal of PMX900

### TO INSTALL


1. With the hose barb to your left, push the trap bottle over the o-ring and up to the bracket.
2. Rotate the trap bottle to your right until it locks into position.

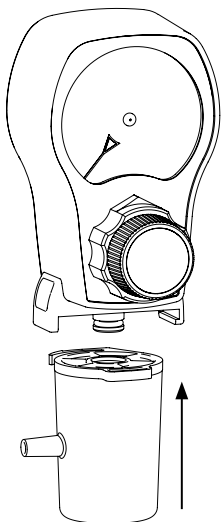

**1**

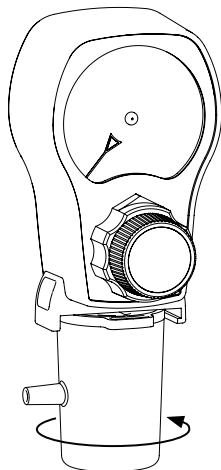


**2**

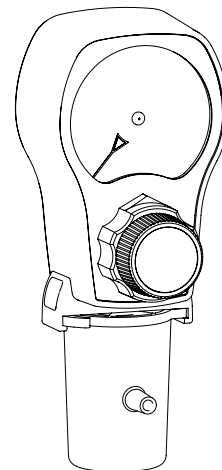


### TO REMOVE


1. Push and hold release tab.
2. Rotate the trap bottle 90° to your left.
3. Pull trap bottle down to remove.

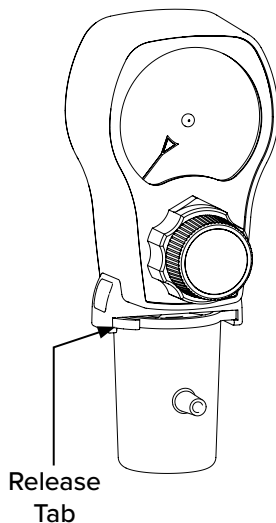

**1**

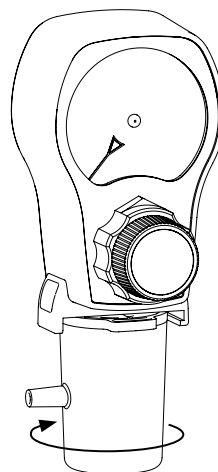

**2**

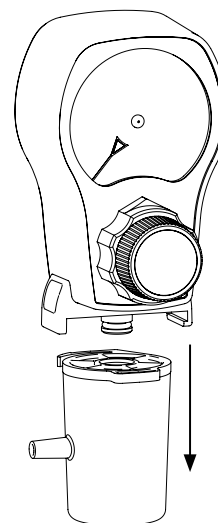

**3**

### Receiving Instructions

When the product is removed from packaging, inspect for damage. If the product is damaged, do not use and contact your dealer or Precision Medical.

### Cleaning

The exterior of the device can be cleaned with wet wipes like Super Sani-Cloth® or similar disinfecting wipes.

### Replacement Part Numbers

Box of 50 Filtered Vacuum Traps: PMX900-50

PMX900 Adapter Kit: PMX901

### Disposal

#### **⚠ WARNING**

After patient use, suction equipment may be contaminated.

If liquid is present in the device, or the device is deemed to be dirty internally, remove from service and discard immediately.

#### **⚠ WARNING**

Dispose of trap in accordance with the local regulations using biohazard precautions